



Using ReeTrakt – It's as Easy as 1-2-3

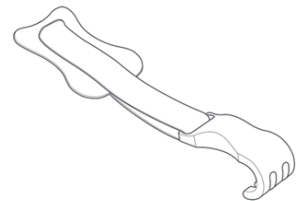

1 Place

Open the blister pack, peel the backing from the adhesive base, and place on a clean, dry part of the patient. The ReeTrakt base will adhere securely.

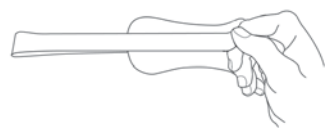

2 Retract

Use the simple hook and loop strap attachment to establish exactly the retraction tension required. Use as many as you need, or as few as two.



3 Dispose

Remove the retraction strap, remove the base from the patient's skin by peeling low and slow, and dispose. You're done – and the low profile ReeTrakt allowed the best visibility possible.

A range of sizes and tips allows use of ReeTrakt for a variety of small incision, soft-tissue procedures.

3PAW SMALL	3PAW MEDIUM	4PAW LARGE	5mm HOOK SHARP	5mm HOOK BLUNT
RTKb-3PAWS	RTK-3PAWM	RTK-4PAWL	RTK-HTIPS5	RTK-HTIPB5

Contact your local sales representative or agent for a full range of available ReeTrakt products.

Insightra ReeTrakt – Ultra-Low Profile Soft Tissue Retraction for a Variety of Surgical Procedures

*A New Family of Ultra-Low-Profile,
Small-Incision, Single Use Retractors.*



*Superior Performance in
a Variety of Applications*

INSIGHTRA
MEDICAL

9200 Irvine Center Drive, Ste. 200
Irvine, CA 92618 USA • www.insightra.com

© 2009 Insightra Medical Inc. USA. All trademarks are owned by their respective companies

CE 0086

Part number: D04-0054-001 Rev. C

ReeTrakt

INSIGHTRA
MEDICAL

ReeTrakt

Latex Free

A completely new, ultra-low profile approach to small incision, soft-tissue retraction that improves surgical access and field of view.

ReeTrakt is a new ultra low profile retraction system that allows better access and visibility to the surgical field, and is designed to minimize tissue damage and eliminate secondary electrocautery burns.

Just pull ReeTrakt from the blister package, peel, stick and you're ready to retract. The simple strap allows rapid and infinitely adjustable retraction. Use as many as you need, or as few as two - ReeTrakt is self-stabilizing. For even more convenience, ReeTrakt can be placed above or below surgical drapes.


ReeTrakt provides stable soft tissue retraction for surgical procedures including –

- AV Fistula
- Smaller Plastics Procedures
- Lumpectomies
- Cut Downs
- Emergency Room
- Small Facial Procedures
- Many More

When the case is done, ReeTrakt is removed by peeling low and slow, then simply disposed of. So not only do you eliminate reprocessing and sterilization time – and associated costs – but there's no risk of cross contamination. No chance of parts going missing. No worn out ratchets or bent tips.

And because ReeTrakt is smaller, easier to use, and significantly less intrusive than any retraction system you've ever tried, it simplifies any procedure and lets your surgical team concentrate on the operation, not the retraction.


Every aspect of ReeTrakt has been optimized for lower profile, less intrusive soft-tissue retraction

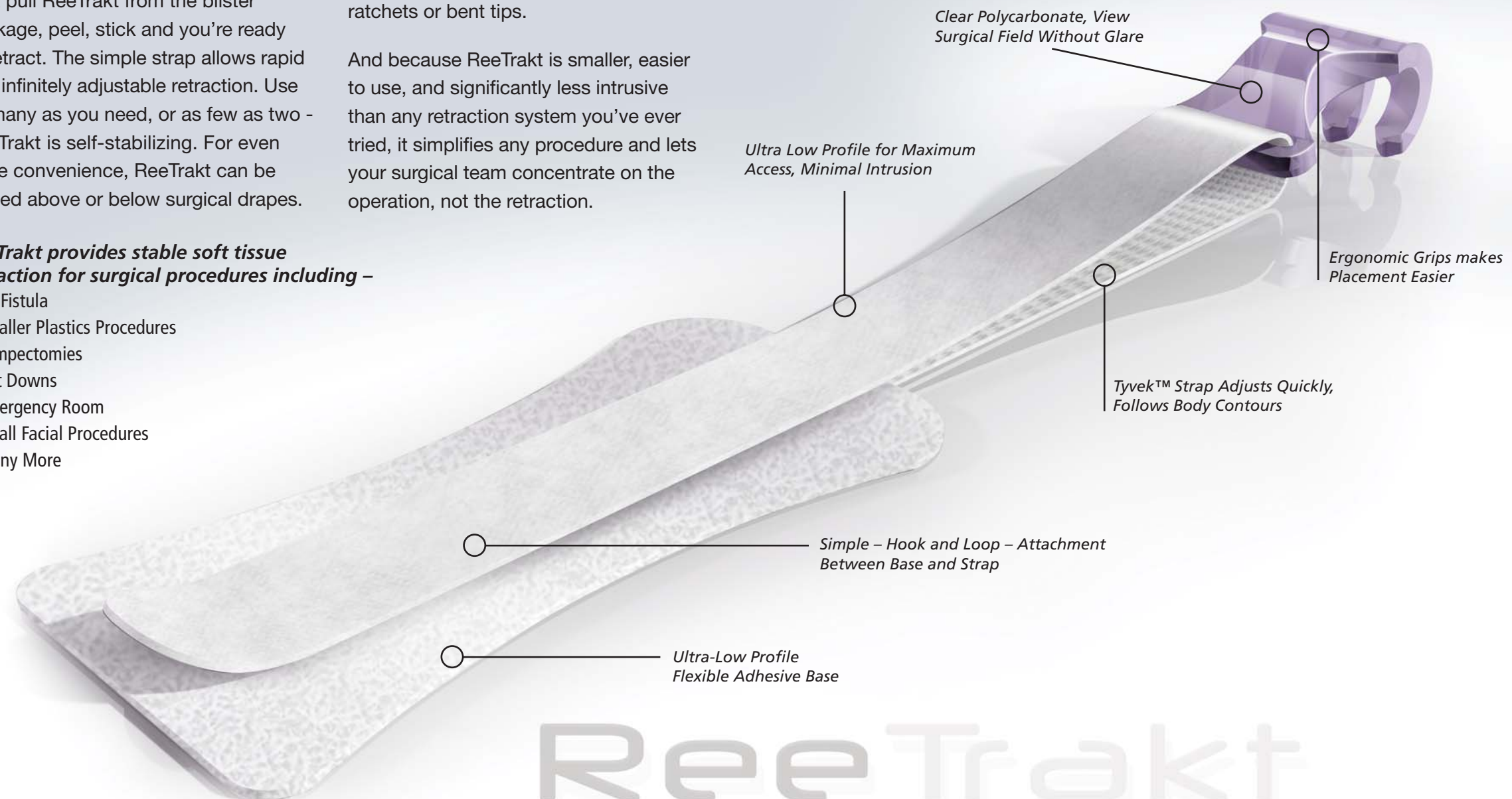

Specialized End Tips designed for specialized procedures.



AV Fistula

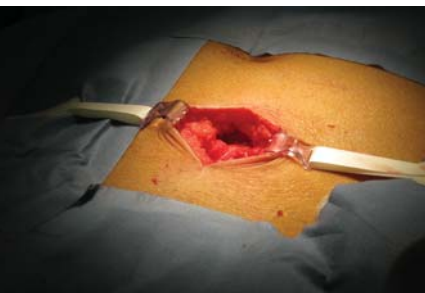

Abdominal



Vaginal Procedures



Carotid Artery

ReeTrakt